

dental CBCT Phantom (BASIC)

Measure imaging performance of dental Cone-Beam Computed Tomography equipment. Perform acceptance and constancy tests with a compact easy-to-use phantom.

The QRM-dentalCBCT Phantom is designed to evaluate the imaging performance of CBCT devices in accordance to international guidelines (e.g. dentalCBCT, Digital Volume Tomography).

The pahntom offers the possibility to assessement all image quality metrics in accordance with national and international standards (e.g. DIN IEC 61223-3-5 or DIN IEC 61223-2-6)

The following image quality metrics can be obtained:

- CT value uniformity
- CT value accuracy
- image noise
- contrast-to-noise ratio
- spatial resolution (3D MTF)

3 defined sections:		
Section I	4 Inserts in water: air, -3% contrast, +3% contrast and bone arranged concentrically	
Section II	homogenious water equivalent slice	
Section III	centrally placed high attenuating sphere	

Technical Specs:


Diameter (xy):	160 mm
Height (z):	150 mm
Body:	$\operatorname{CTWater}^{\mathbin{\circledR}}$
Sphere:	Al, Ø 12 mm
Inserts:	Ø 13 mm, L 25 mm

An automated evaluation software (ImpactIQ Wizard) is available seperately or as a package with the


Please note: if using the software, your systems FOV has to completely cover the phantom in height, otherwise a correct automated evaluation is not possible.Please ask for more information: info@qrm.de


References:

A quality assurance framework for the fully automated and objective evaluation of image quality in cone-beam computed tomography Ch Steiding, D Kolditz, WA Kalender; Medical Physics, 41, 031901 (2014)

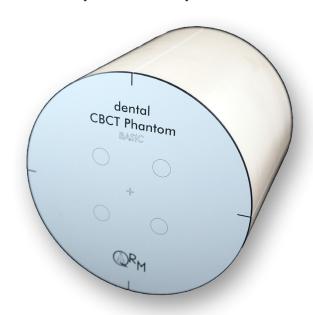


dentalCBCT-Phantom BASIC

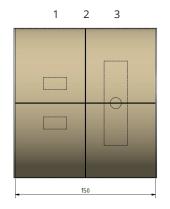

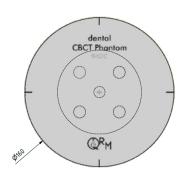

cross sections of phantom

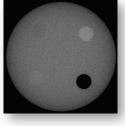

1 scaling: bone water +3%: -3%

2 noise: noise homogeinety





3 MTF: 12 mm sphere

in-plane reconstruction of X-ray CBCT scans 吕先生: 15920060912(微信同号) 0755-22934005(座机)

深圳市福田区八卦二路八卦岭工业区615栋419